


Saliva Sample Collection Instructions

For In Vitro Diagnostic Use

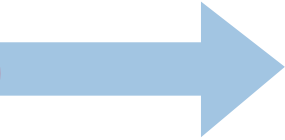

Collection precautions:

Ensure the sponge tip does NOT come into contact with any surface prior to collection.

Donor should not eat, drink, smoke, or chew gum for 30 minutes before collecting oral sample.

Intended use: This product is designed for the collection of human oral samples.

Contents: Kit contains liquid.

Warning and precautions: Choking hazard; caution should be used when inserting sponge into the mouth.

Wash with water if liquid comes in contact with eyes or skin. Do not ingest.
View the MSDS at www.dnagenotek.com.

Storage: 15–30°C

Summary and explanation of the kit:

ORAcollection is a self-collection kit that provides the materials and instructions for collecting human oral samples.

For donor collection instructions in other languages, visit: http://dnagenotek.com/DNA_Genotek_Support_Lit_UI_OC-100.html

For more information about ARUP Genetics, including ordering instructions, please visit: www.aruplab.com/genetics

ENGLISH

Procedure for oral sample collection kit:

Illustration of a person's mouth with a blue sponge tip being inserted into the lower gum area. A red dashed circle highlights the tip's position. A red arrow points to the tip. A red double-headed arrow indicates a back-and-forth motion. The text "10 x" is shown. A circular inset below shows a close-up of the sponge tip in the mouth.	<p>Open package and remove collector without touching sponge tip. Place sponge as far back in the mouth as comfortable and rub along the lower gums (see close-up image) in a back and forth motion. Gently rub the gums 10 times. If possible, avoid rubbing the teeth.</p>
Illustration of a person's mouth with the sponge tip on the opposite side of the lower gum. A red dashed circle highlights the tip's position. A red arrow points to the tip. A red double-headed arrow indicates a back-and-forth motion. The text "10 x" is shown.	<p>Gently repeat rubbing motion on the opposite side of the mouth along the lower gums for an additional 10 times.</p>
Illustration of hands unscrewing the blue cap from the collection tube. A red circle with a slash is over a cigarette, indicating no smoking. The text "3" is shown.	<p>Hold the tube upright to prevent the liquid inside the tube from spilling. Unscrew the blue cap from the collection tube without touching the sponge.</p>
Illustration of hands inserting the sponge tip into the collection tube and closing the cap. The text "4" is shown.	<p>Turn the cap upside down, insert the sponge into the tube, and close cap tightly.</p>
Illustration of hands shaking the capped collection tube. A red curved arrow indicates the shaking motion. The text "10 x" is shown.	<p>Invert the capped tube and shake vigorously 10 times.</p>



NATIONAL REFERENCE LABORATORY

500 Chipeta Way | Salt Lake City, UT 84108 | Phone: (800) 522-2787 | www.aruplab.com

ARUP is a nonprofit enterprise of the University of Utah and its Department of Pathology.

IOG-CI-49295, Rev. 0 | April 2018 | Page 1 of 1