




## ALZHEIMER'S DISEASE (AD) MARKERS, CSF (TEST #3017653)

### Cerebrospinal Fluid (CSF) Specimen Collection Instructions


**Step 1** Perform lumbar puncture using the gravity drip collection method.

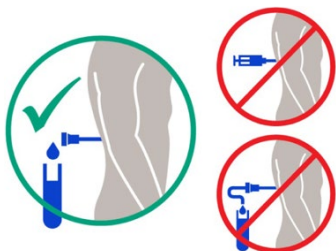


**Step 2** Do not use the first 2 mL of CSF for AD biomarker measurement.



**Step 3** Collect at least 2 mL of CSF directly into CSF AD biomarker tube.

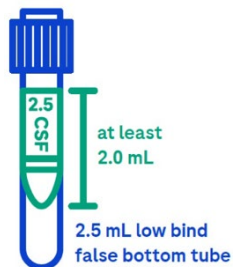

Do not process the CSF sample before transport (e.g., no mixing/inverting, no aliquoting, no centrifugation).

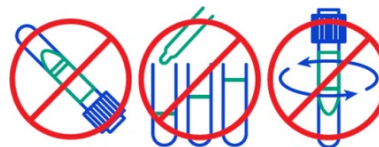

**Step 4** Ship frozen or refrigerated.

