# Contents Ref. 149995

## ARUP Supply # 52059

- 1 sealable bag for the preservation of the tube (Bag A)
- 1 clear tube with green cap containing liquid (AlcorFix™)
- 1 green scoop assembly

### Instructions for Use:


**1**

a) Collect the stool sample in a clean, dry receptacle (a bedpan, plastic bag, or plastic wrap for example) which does not have residue from soap or disinfectants.

b) **Do not use stool samples which have been contaminated with water, urine, or toilet paper.**


**2**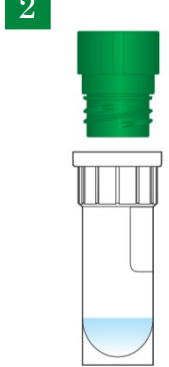

Unscrew the lid of the tube containing the liquid and place on flat surface where it will not be easily knocked over.

Discard the cap.

**3**

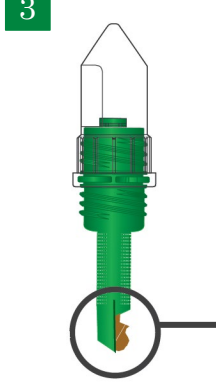a) Pick up a green/clear scoop assembly, holding the smaller tube by the conical end.

b) Use the green scoop to collect:

Liquid Stool: two level scoops

Solid Stool: one heaped scoop of stool (see example in figure below).

Sample from the center of the stool

 x 2  
Liquid stool

 Heaped scoop x 1  
Solid stool

**4**


a) Place the scoop assembly containing the stool sample into the tube containing the liquid (do not separate the spoon from the tube attached).

b) Screw together as **tightly as possible—until you feel a “click”**—that completes the seal. Leave fully assembled as shown in this diagram

c) Fill out the patient information on the loose label and affix to the top conical portion of the tube.

d) Fill out patient information on the tube label.

e) Place the assembled tube inside Bag A. Store at room temperature; do not refrigerate.

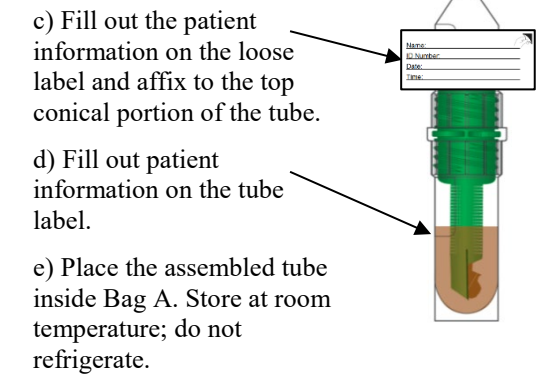**5**

Wash your hands thoroughly with soap and water.

**6** If it is necessary to collect additional stool samples on different days, follow your physician’s instructions and repeat steps 1–5 for the additional samples using a new Patient Care pack.

**7** Return sample(s) to your lab or healthcare provider as instructed.

**Important**

**Do NOT drink.**

The liquid in the enclosed tube is toxic (AlcorFix™ solution). Do not drink. Keep out of reach of children.

For a poison emergency in the United States, call the American Association of Poison Control Centers at (800) 222-1222.

In the event of contact with skin, eyes or mucous membrane, wash immediately with plenty of cold water and seek medical attention.

APACOR

Parasep® SF Patient Care Single Vial  
 Sample Collection Instructions for Use

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