

Shipping Time-Sensitive Refrigerated Specimens with a Stability of 48 Hours

In our ARUP Test Directory, 'Time Sensitive' refers to specimens that are refrigerated and must be received at ARUP within 48 hours from collection. A test that is considered Time Sensitive has a stopwatch icon located with the test information in the ARUP Test Directory.

IMPORTANT: This new guide only applies to refrigerated specimens with a 48-hour stability.

1. Fill out required information on bag.
 2. Place time-sensitive label over the bag's barcode.
 3. Place smaller barcode label on bag's tear-off portion for your records.
 4. Place rack, containing time-sensitive specimens only, into refrigerated bag.
 5. Insert time-sensitive specific packing list in the bag sleeve.
 6. For orders placed through an Interface, please stamp your packing list with "Time Sensitive" (eSupply: Item # 56785).
- Couriers will pick up all bags at once, package, and ship specimens.
- NOTE: Order time sensitive labels through eSupply: Item # 56529.

